JAPANESE

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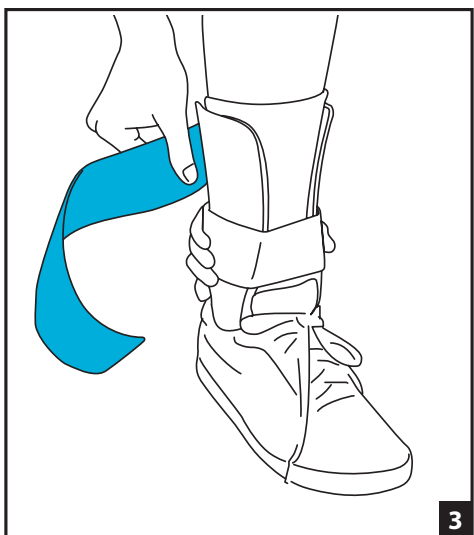
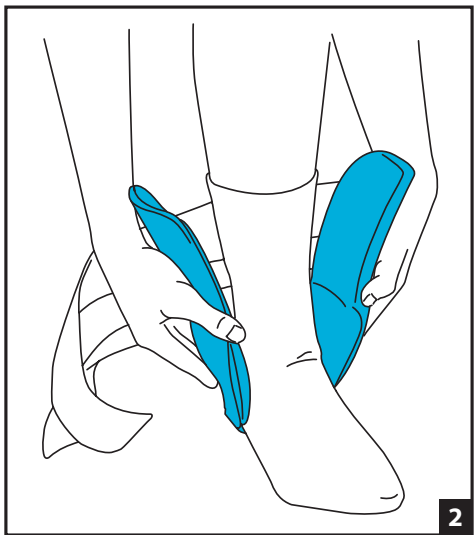
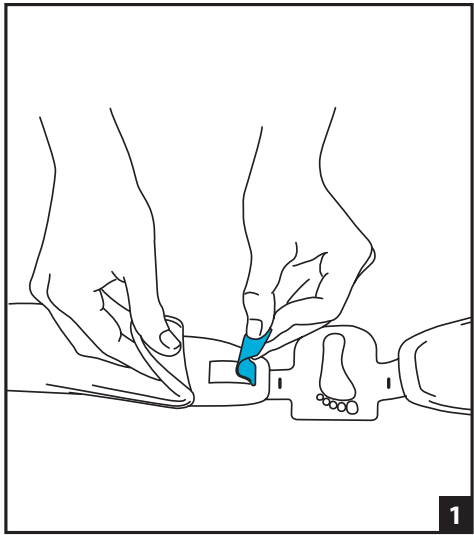
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

Intended Use / Indications: The Air-Stirrup provides support to prevent further ankle injury by effectively limiting inversion and eversion.

Contraindications: NA

WARNINGS AND PRECAUTIONS:

Like all lower extremity devices, such as casts or braces, patients without sensation (i.e. post-op anesthesia, neuropathies, etc.) should be monitored frequently for "hot spots", skin irritation or wound management.

Always wear an absorbent sock or ankle wrap when using the Air-Stirrup Ankle Brace.

APPLICATION INFORMATION:

1. Adjust heelpad width

- Open brace and lift bottom of aircells until heel straps are fully exposed.
- Peel up heel straps and adjust heelpad width for a snug fit (Fig. 3).
- Firmly press heel straps and aircells back in place.

2. Apply brace

- Put on absorbent sock or ankle wrap.
- Place round edge of heelpad under heel.
- Align brace sides with ankle (Fig. 4).

3. Secure straps

- Secure straps from bottom to top.
- Put on lace shoe or sneaker.

4. Adjust brace

- Squeeze brace sides together with one hand.
- Tighten straps from bottom to top with other hand (Fig. 5).
- Tighten for comfortable support.

ANKLE BRACE FIT ADJUSTMENT

Ankle Brace aircells are preinflated and normally do not require adjustment.

If you experience pinching or uneven pressure:

- Reposition brace or,
- Readjust heelpad and straps. Use lines on heelpad as a reference.

If condition persists, adjust aircell volume (**too much air will reduce support**):

- Unfold valve on top of aircell.
- Insert inflation tube into valve 1 1/2 inches.
- Remove a **small amount** of air by gently squeezing aircell or,
- Add a **small amount** of air by blowing into tube.
- Squeeze valve flat just below tube, and slowly remove tube.
- Fold valve.

HIGH ALTITUDE:

AT HIGH ALTITUDES THE AIRCELLS WILL EXPAND BEYOND THEIR OPTIMAL LEVEL, AND AIR WILL HAVE TO BE REMOVED FROM THE AIRCELLS (SEE ANKLE BRACE FIT ADJUSTMENT). WHEN FLYING, READJUST STRAPS TO A COMFORTABLE PRESSURE.

CARE

The entire Ankle Brace can be hand washed in lukewarm water with mild soap and air dried.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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