SVENSKA

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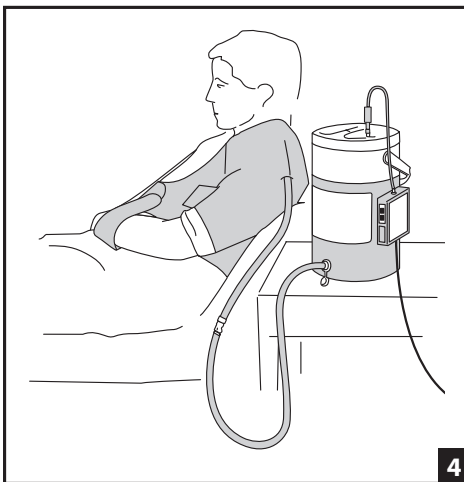
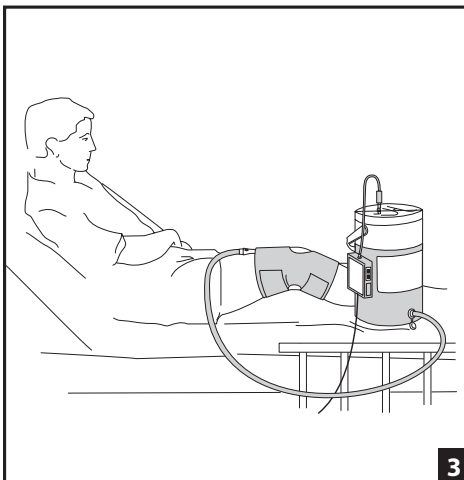
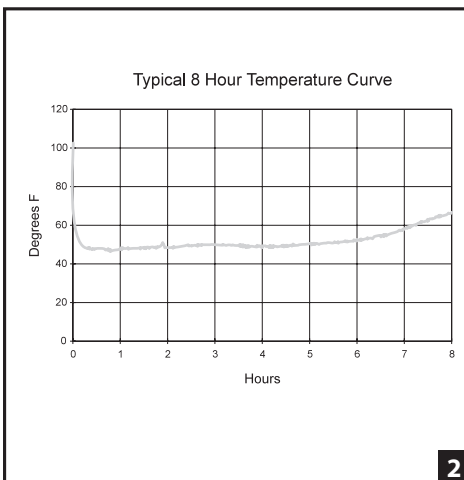
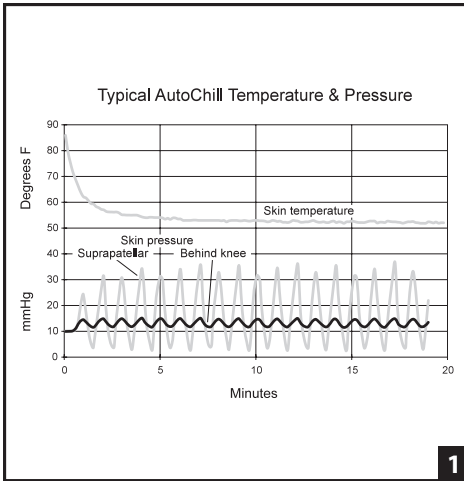
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS:
For use with Aircast® Cryo/Cuff™ cold and compression dressings. **PROVIDES INTERMITTENT COMPRESSION FOR USE IN CRYOTHERAPY.**

CONTRAINDICATIONS:
Cryotherapy should not be used on persons with Raynaud's or other vasospastic disease, cold hypersensitivity, decreased skin sensitivity, or compromised local circulation. Observe all Cryo/Cuff Contraindication, Caution, and Warning statements.

- WARNINGS AND PRECAUTIONS:**
- Risk of explosion if used in the presence of flammable anesthetics.
 - If bandages or dressings become moist or wet, discontinue use of the AutoChill System and immediately contact your healthcare professional.
 - Patient's skin should be observed frequently due to individual sensitivity and susceptibility to cold.
 - Do not allow fluid to come in contact with the air pump.
 - Unplug pump before cleaning to prevent electrical shock.

OPERATION:
During AutoChill operation, the electronic pump automatically cycles compressing the air in the cooler to force chilled water into the cuff. This repetitive cycling maintains continuous cold and applies soothing pulsating pressure. (Figs. 1 and 2). If the cooler is tipped from its upright position, the check valve on the tube assembly will close off the air tube, preventing water from entering the pump and causing pump damage. The excess water will be vented from the air tube outside of the system. Once the cooler is up-righted, the system will self-clear.

- APPLICATION INFORMATION:**
1. **Prepare cooler** (See Cryo/Cuff™ instruction sheet.)
 2. **Apply the EMPTY cuff** (See Cryo/Cuff instruction sheet.)
 3. **Fill cuff** (See Cryo/Cuff instruction sheet.)
- When using the AutoChill System, the cooler must remain connected to cuff.**

4. **Correctly position cooler**
Place the cooler on a bed or table. The base of the cooler should be even with the bottom of the cuff. **Do not raise cooler above the cuff when using the AutoChill System.**
5. **Connect air pump**
Connect clear air tube from pump to fitting on cooler lid, with check valve at cooler end. Connect wall adapter power cord to bottom of pump then plug wall adapter into outlet (Fig. 3). Press pump power switch to begin operation. If using the Shoulder Cryo/Cuff, press the pump mode switch and the "Shoulder Only" light will illuminate (Fig. 4).

THE AUTOCHILL SYSTEM CAN BE USED WITH ALL AIRCAST CRYO/CUFF PRODUCTS.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**For single patient use only.
RX ONLY
LATEX FREE**

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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