

# A2™ Wrist Brace

AIRCAST®  
LLC

wrist support (05WS, 05WM, 05WL)

The Aircast A2 Wrist Brace is designed to provide support for wrist injuries (ligament instability, sprain or muscle strain), carpal tunnel syndrome, post-operative use, and post-removal of casting or splint. Easy to apply, the A2 Wrist Brace can be worn during most daily activities.

### The A2 Wrist Brace features and benefits:

- Dual stabilizers restrict wrist movement while permitting full finger function
- Adjustable straps and three sizes ensure personalized fit
- Contoured shape and breathable fabric promote comfort




## Instructions:

### 1 Prepare brace

Unfasten straps. Middle strap and arm strap should remain threaded through black rings on brace.

### 2 Apply brace

Align brace with green fabric toward thumb and thin strap (hand strap) toward fingers. Insert hand into brace (Fig. 1). Pull brace up wrist until end of brace is just past knuckles (fingers should bend freely at knuckles) (Fig. 1). Secure brace straps in order: middle strap (Fig. 2), hand strap (Fig. 3), and arm strap (Fig. 4). Tighten all straps until snug and comfortable. To improve comfort, straps may be trimmed with scissor.

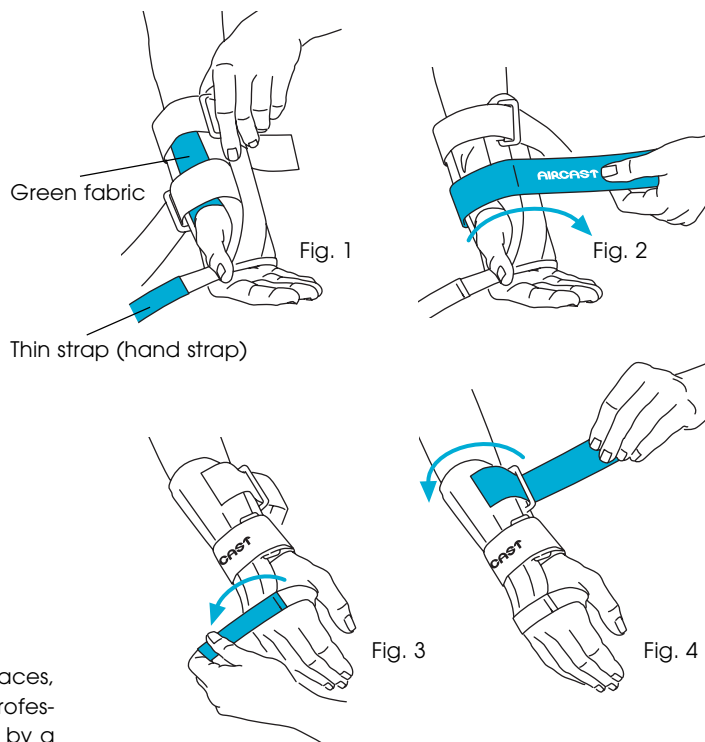

### CAUTION

This product, like all upper extremity immobilizers such as casts or braces, should only be applied with the advice and consent of a medical professional. Appropriate use of this product **requires** that it be dispensed by a medical professional for patients with upper extremity neuropathies (i.e. loss of sensation) or other vascular problems that may be aggravated by the brace. Medical professionals should instruct patients to check for skin irritation or discoloration of fingers with use. If skin irritation or discoloration occurs, patients must be instructed to remove the brace and recontact a medical professional immediately.

### CARE

The A2 Wrist Brace can be hand washed in cold water with mild soap and air dried. **Remove dual stabilizers before cleaning:** locate stabilizer opening in fabric on the inside of brace and slide stabilizer out opening.

### LATEX

This product is latex-free.

### WARRANTY POLICY

Aircast will provide a replacement to the original purchaser should the A2 Wrist Brace become unserviceable for any reason for a period of 90 days from date of sale, provided the defective part is returned to Aircast for analysis.

\*A2™ is a trademark of Aircast LLC, Summit, New Jersey, USA.

EU Representative

**AIRCAST®**  
EUROPE GmbH

Georg-Wiesböck-Ring 12  
D-83115 Neubeuern  
Tel: 00800-00 85 24 33 (free phone)  
0049 8035 87 99 50  
Fax: 00800-00 85 40 15 (free fax)  
0049 8035 87 99 87

Manufacturer  
**CE AIRCAST®**  
LLC

92 River Road  
Summit, NJ/USA 07902-0709  
Tel: (800) 526-8785  
Tel: (908) 273-6349  
Fax: (800) 457-4221  
Fax: (908) 273-1060  
www.aircast.com

R 1/11/06  
05W100B  
0510

# A2™ Wrist Brace with Thumb Spica

AIRCAST<sup>®</sup> LLC

wrist and thumb support (05WTS, 05WTM, 05WTL)

The Aircast A2 Wrist Brace with Thumb Spica is designed to provide support for wrist and thumb injuries (ligament instability, sprain or muscle strain), carpal tunnel syndrome, post-operative use, and post-removal of casting or splint. Easy to apply, the A2 Wrist Brace with Thumb Spica can be worn during most daily activities.

### The A2 Wrist Brace features and benefits:

- Dual stabilizers restrict wrist movement while permitting full finger function
- Adjustable straps and three sizes ensure personalized fit
- Contoured shape and breathable fabric promote comfort
- Thumb Spica holds thumb securely in place




## Instructions:

### 1 Prepare brace

Unfasten straps. All straps, except thumb strap, should remain threaded through black rings on brace.

### 2 Apply brace

Align brace with green fabric on top of hand and thumb spica toward thumb. Insert hand into brace with thumb inserted in thumb spica (Fig. 1). Pull brace up wrist until end of brace is just past knuckles (all fingers should bend freely at knuckles except thumb). Secure brace straps in order: middle strap (Fig. 1), hand strap (Fig. 2), and arm strap (Fig. 3). Wrap thumb strap around thumb spica and secure (Fig. 4). Tighten all straps until snug and comfortable. To improve comfort, straps may be trimmed with scissor.

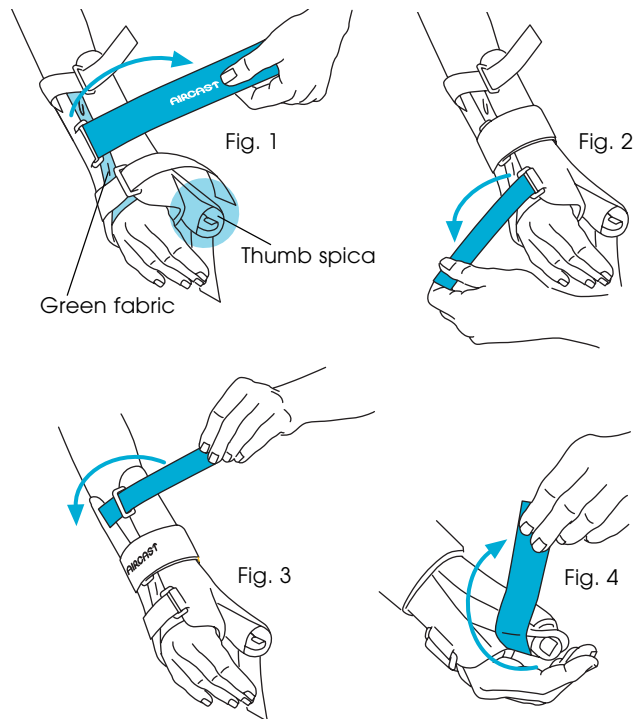

### CAUTION

This product, like all upper extremity immobilizers such as casts or braces, should only be applied with the advice and consent of a medical professional. Appropriate use of this product **requires** that it be dispensed by a medical professional for patients with upper extremity neuropathies (i.e. loss of sensation) or other vascular problems that may be aggravated by the brace. Medical professionals should instruct patients to check for skin irritation or discoloration of fingers with use. If skin irritation or discoloration occurs, patients must be instructed to remove the brace and recontact a medical professional immediately.

### CARE

The A2 Wrist Brace with Thumb Spica can be hand washed in cold water with mild soap and air dried. **Remove dual stabilizers before cleaning:** locate stabilizer opening in fabric on the inside of brace (for hand stabilizer) and on outside of thumb (for thumb stabilizer) and slide stabilizer out opening.

### LATEX

This product is latex-free.

### WARRANTY POLICY

Aircast will provide a replacement to the original purchaser should the A2 Wrist Brace with Thumb Spica become unserviceable for any reason for a period of 90 days from date of sale, provided the defective part is returned to Aircast for analysis.

\*A2™ is a trademark of Aircast LLC, Summit, New Jersey, USA.

EU Representative

**AIRCAST<sup>®</sup>**  
EUROPE GmbH

Georg-Wiesböck-Ring 12  
D-83115 Neubeuern  
Tel: 00800-00 85 24 33 (free phone)  
0049 8035 87 99 50  
Fax: 00800-00 85 40 15 (free fax)  
0049 8035 87 99 87

Manufacturer  
**CE AIRCAST<sup>®</sup>** LLC

92 River Road  
Summit, NJ/USA 07902-0709  
Tel: (800) 526-8785  
Tel: (908) 273-6349  
Fax: (800) 457-4221  
Fax: (908) 273-1060  
www.aircast.com

R 1/11/06  
05W100B  
0510