JAPANESE

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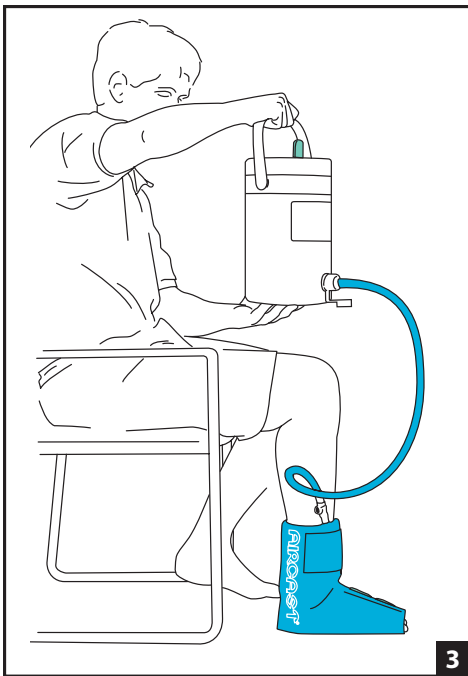
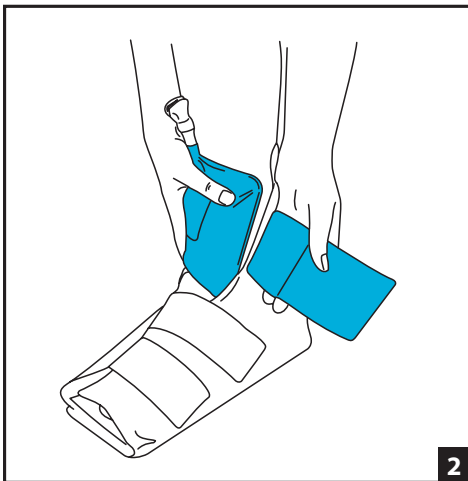
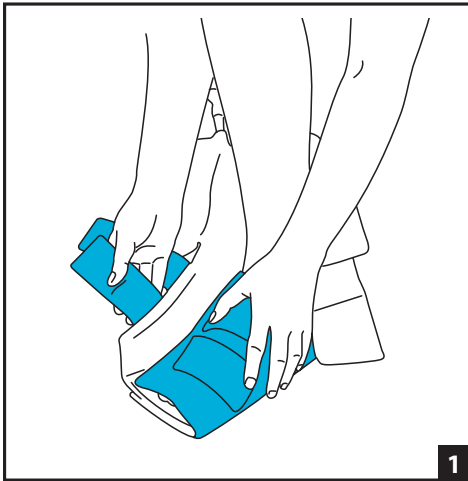
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS:
The Aircast Cryo/Cuff combines focal compression with cold to provide optimal control of swelling, edema, hematoma, hemarthrosis, and pain.

FREQUENCY OF USE:
The length of use and frequency of use of the Cryo/Cuff are determined by the healthcare professional depending on individual patient's needs.

CONTRAINDICATIONS:
Cryotherapy should not be used on persons with Raynaud's or other vasospastic disease, cold hypersensitivity, decreased skin sensitivity, or compromised local circulation.

WARNINGS AND PRECAUTIONS:
• An Ankle Cryo/Cuff with an AutoChill system should only be used in a medical facility or clinical environment with direct healthcare provider supervision.
• When using the Ankle Cryo/Cuff with the AutoChill system, always apply a heavy cotton sock or bandaging to prevent skin temperature from falling below 50°F (10°C).
• Do not use an elastic wrap in conjunction with the Cryo/Cuff.
• Dressings used under the Cryo/Cuff should be applied lightly.
• Patient's skin should be observed frequently due to individual sensitivity and susceptibility to cold.

• When filling the Cryo/Cuff, do not raise the cooler higher than 15 inches (38 cm) above the cuff to avoid excessive pressure.
• Reduce pressure with any sense of discomfort, numbness or tingling of the limb.
• The Cryo/Cuff is designed for single patient use only and may be used on the same patient for the length of treatment. The device should be cleaned if it is used on the same patient for an extended period of time (see "Cleaning Instructions" for cleaning information).
• Federal law restricts this device to sale by or on the order of a licensed healthcare professional.

APPLICATOR INFORMATION:

- Prepare cooler** - Connect blue tube to cooler. Add water to line inside cooler (additional water, up to 5 inches (5 cm), may be needed to completely fill cuff), then add ice. For 6 to 8 hour treatment, add ice to top of cooler. Lay the insulation disk on top of ice. Attach cooler lid snugly. Allow 5 minutes with occasional shaking to chill water.
- Apply the EMPTY cuff** - Close bottom flaps under foot. Place foot in cuff with heel against back of cuff. If necessary, adjust cuff width by repositioning bottom flaps. Wrap foot cover around foot and secure with straps (Fig. 1). Wrap ankle cover around ankle and secure strap (Fig. 2).
- Fill cuff** - Connect blue tube to cuff. Open air vent on cooler lid and raise cooler no more than 15 inches (38 cm) above the cuff (Fig. 3). (If the cooler is raised more than 15 inches (38 cm), the cuff will increase in volume and weight.) Hold raised cooler for about 30 seconds while cuff fills. Close air vent on cooler lid. If not using an Aircast AutoChill System, the cooler can now be disconnected from cuff by pressing the metal tab on the quick-disconnect while cooler is raised.
- Rechill water** - At first, rechill the water in the cuff after 15 or 30 minutes, then once an hour or as needed. Reconnect the blue tube to cuff, lower the cooler below cuff, and the warmed water will drain from the cuff into the cooler. Allow a minute or two for the water to mix with the ice and rechill, then raise the cooler and repeat the filling process (see step 3).

CLEANING INSTRUCTIONS:
After use, completely drain water from cuff, tube, and cooler. (To drain tube, elevate tube while pressing tip of quick-disconnect.) Store cooler with top off to allow drying. Periodically clean cuff, tube, and cooler using a few ounces of liquid soap added to hot water in cooler. Recycle soap/water mixture between cooler and cuff a few times, then repeat with warm water only. Rinse completely.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

For single patient use only.
RX ONLY
LATEX FREE

NOTICE: WHILE EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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