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HUMERAL STABILIZING SYSTEM

HUMERAL STABILIZING SYSTEM APPLICATION INSTRUCTIONS

Before applying the brace, please read these instructions completely and carefully.

Correct application is vital to the proper functioning of the brace.

INDICATIONS: • Stable Humeral fractures • Stable Supracondylar fractures • ORIF Humeral fractures • Bicep/tricep repairs
This system can be used in acute treatment of humeral fractures.

Its design assists in reducing some of the initial discomfort common during the acute stages of injury
This product was designed to complement the variety of medical treatments common to the above afflictions.
The frequency and duration of use should be determined by your prescribing Healthcare professional.

A Begin with the patient seated, leaning slightly forward. While supporting the forearm at the wrist and hand, allow the upper arm to hang forward from the body.

B Align the humeral cuff so that the elbow hinge is in line with the lateral epicondyle and the sheepskin pad sits over the acromion. Wrap the humeral cuff around the upper arm making sure the shell and liner overlap 1"-1 1/2" (2.5 cm-3.8 cm). The shell may be trimmed to accommodate smaller limb circumferences. Finally, secure the straps ensuring a snug fit.

IMPORTANT: Do not over-tighten. Monitor distal circulation at regular intervals.

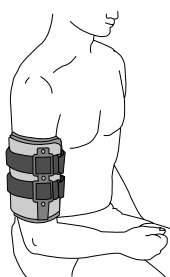
C Raise the forearm portion of the brace placing the wrist and hand into the cuff as shown. The thumb should rest comfortably around the hand grip bar. Secure the strap at the wrist.

D Attach the shoulder strap from the acromion pad at the shoulder to the hand grip bar as shown. Position the neck pad to rest comfortably at the back of the neck.


E A 2lb (.9 kg) weight is included to provide slight traction to the bone during healing. Use of the weight is optional, and purely at the discretion of the treating physician. To apply, wrap the weight around the forearm just distal to the elbow as shown.

Note: The cuff weight may be cut down the middle to reduce the weight in half.

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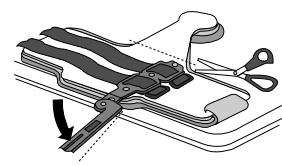
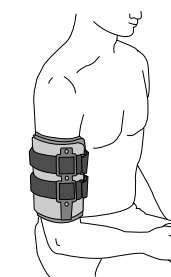
© dj Orthopedics, LLC

BREAKDOWN INSTRUCTIONS

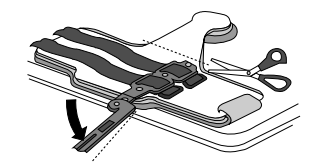Once the fracture is partially healed and stable, the Humeral Stabilizing System may be broken down to consist of the humeral cuff portion only.

Find the score mark located on the metal upright, just above the elbow pivot attachment. While stabilizing the upper cuff against a solid surface, bend the bar at the score mark. The bar will break at the score mark after a few bends have been made.

Note: After breaking the bar, you may wish to file the sharp surface at the break to prevent injury.

Using scissors, remove the upper part of the shoulder cuff as shown.

USE AND CARE

- The cuff liner and sheepskin pad may be removed for cleaning or replacement.
 - To clean liners, hand wash using a mild soap and then air dry.
- Always consult with your physician or therapist before making changes to the brace.

WARNING: THIS PRODUCT IS TO BE USED UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. THIS DEVICE IS NOT INTENDED FOR UNSUPERVISED PUBLIC USE. IF YOU EXPERIENCE ANY PAIN, SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTIONS WHILE USING THIS PRODUCT, CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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dj Ortho and DonJoy are trademarks of dj Orthopedics, LLC