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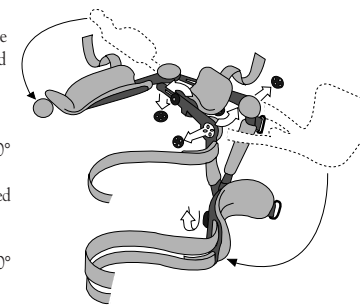
SCOI SHOULDER BRACE APPLICATION INSTRUCTIONS

Before applying the brace, please read these instructions completely and carefully.
Correct application is vital to the proper functioning of the brace.


INDICATIONS: • Rotator cuff repairs • Capsular shifts • Bankhart procedures • Prosthetic shoulder replacements
*This product was designed to complement the variety of medical treatments common to the above afflictions.
The frequency and duration of use should be determined by your prescribing Healthcare professional.*

For convenience and post-op application ease, it is recommended to pre-fit this brace to the patient before surgery.

A Release hinges at the shoulder, elbow, and upper arm cuff and position the two shoulder hinges between 60° and 90° of abduction. The brace can be adapted for the right or left side by rotating the forearm portion 180° at the elbow joint. The shoulder hinge pads are removable and should be set up with the sheepskin pad posterior, and foam pad anterior. Open all straps, folding the ends of the strap back onto themselves for easier handling. Loosen, but do not remove the various knobs at the locations shown.

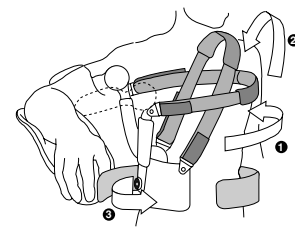
C If the patient is in a supine position, move the patient into an upright sitting position and pass the shoulder strap up and over the uninvolved shoulder and chest strap across the upper back. Lay the waist strap across the table, behind the patient just above the hip level.


D Return the patient to a supine position and affix all three straps in the following order: 1. shoulder strap (attaching to the hip plate), 2. chest strap (around the back and under the opposite arm, attaching to the front upper ring on the upright bar, and 3. the waist strap crossing above the stomach region. Make sure all straps are snug. Use the color coded dots on the straps to assist in locating the appropriate attachment points.

Note: The hook pieces attached to the shoulder and chest strap pads are used to attach the two straps together where they cross in the front and back of the chest. This will help keep the straps in position.

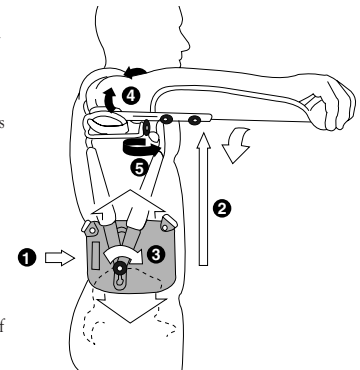B Adjust the torso bars to patient height by first positioning the large hip plate so that the foam wedge on the inside of the plate rests on the iliac crest. Loosen the knob on the plate and adjust the height of the torso bar so that the top shoulder hinge is aligned with the glenohumeral joint. The patient's arm should rest comfortably in the humeral and forearm cuffs of the brace. Tighten the height adjustment knob to one of the four set height positions.

E Loosen the two adjustment knobs located on either side of the upperarm support bars. Adjust the humeral portion of the brace so that the elbow pad/hinge is centered under the patient's elbow. Re-tighten the two adjustment knobs. Secure the humeral cuff straps. Loosen the two knobs on the undersurface of the forearm bar. Slide the forearm cuff until the cockpit portion of the support rests comfortably in the palm. The cuff may also be rotated to accommodate for supination or pronation. Re-tighten the knobs and secure the forearm strap.

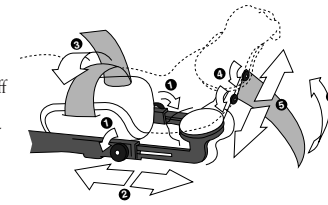F With the arm held in the desired immobilizing position, lock all hinges.

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
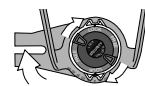
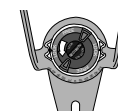
© dj Orthopedics, LLC

INSTRUCTIONS FOR HINGE ADJUSTMENT

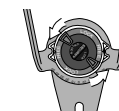
Loosen the small black security knob 1-1 1/2 turns counterclockwise.

Rotate the black adjustment knob until it clicks into the free position.

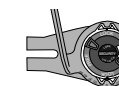Once the hinge is in the desired position of abduction, rotate the black adjustment knob until it clicks into the lock position.

Tighten the security knob clockwise to ensure the hinge is secure.

USE AND CARE

- The patient should be instructed to regularly check the tightness of all the knobs.
- The hip plate is heat formable if a slight change of shape is required.
- Liners may be hand washed in cold water. Air dry.

Always consult with your physician or therapist before making changes to the brace.

WARNING: THIS PRODUCT IS TO BE USED UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. THIS DEVICE IS NOT INTENDED FOR UNSUPERVISED PUBLIC USE. IF YOU EXPERIENCE ANY PAIN, SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTIONS WHILE USING THIS PRODUCT, CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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dj Ortho and DonJoy are trademarks of dj Orthopedics, LLC