JAPANESE

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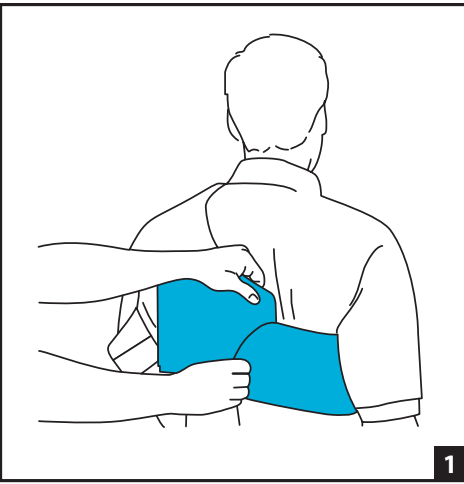
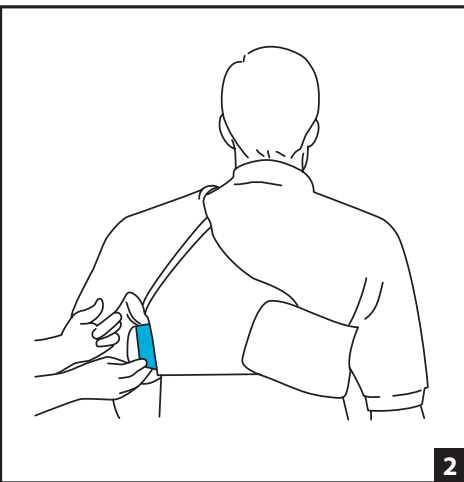
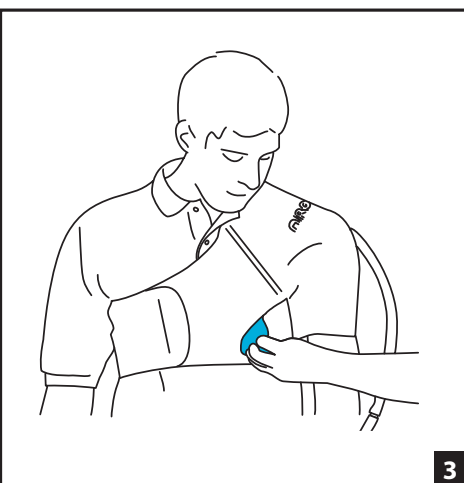
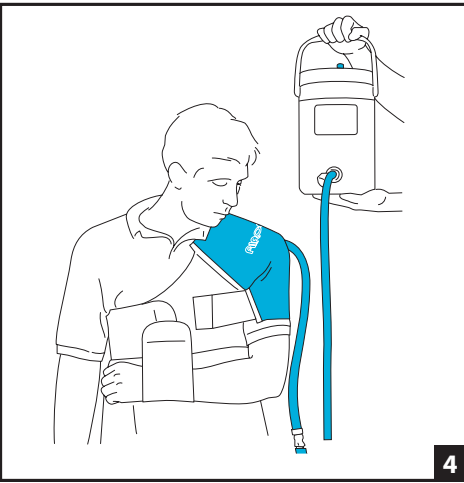
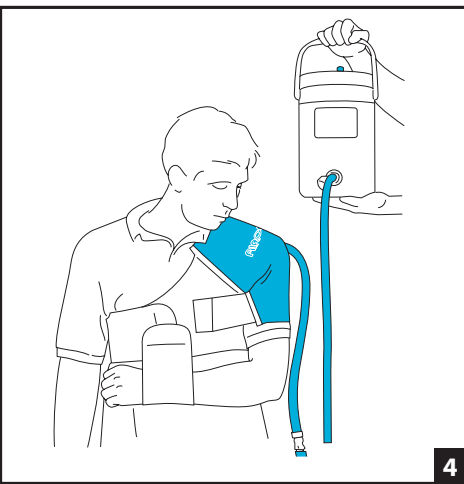
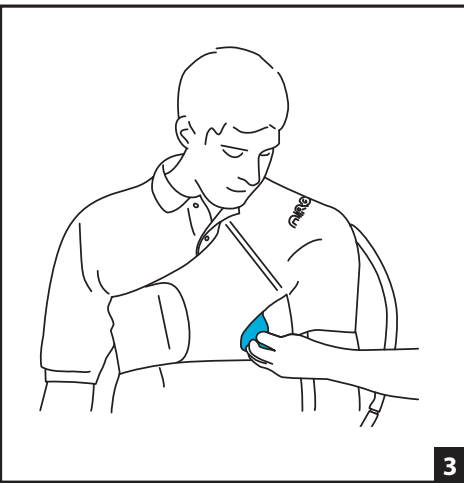
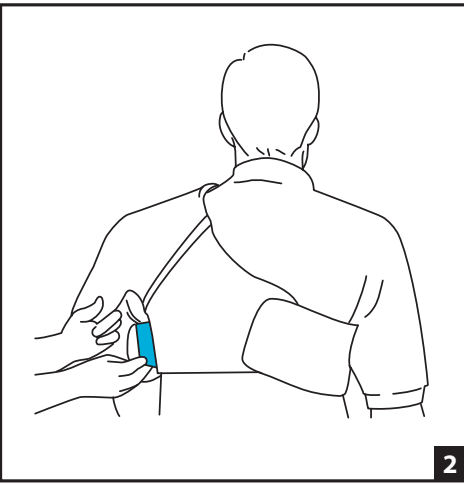
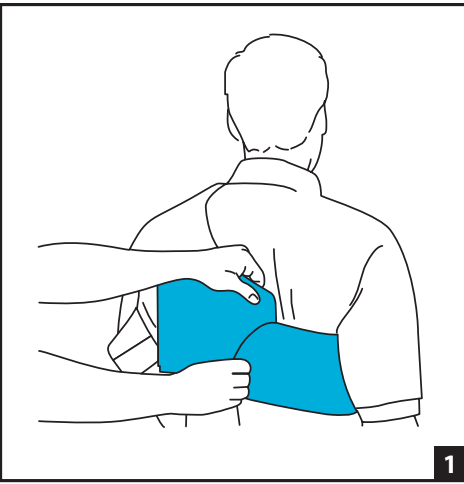
DJO LISTED

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS:

The Aircast Cryo/Cuff combines focal compression with cold to provide optimal control of swelling, edema, hematoma, hemarthrosis, and pain.

FREQUENCY OF USE:

The length of use and frequency of use of the Cryo/Cuff are determined by the healthcare professional depending on individual patient's needs.

CONTRAINDICATIONS:

Cryotherapy should not be used on persons with Raynaud's or other vasospastic disease, cold hypersensitivity, decreased skin sensitivity, or compromised local circulation.

WARNINGS AND PRECAUTIONS:

- Do not use an elastic wrap in conjunction with the Cryo/Cuff.
- Dressings used under the Cryo/Cuff should be applied lightly.
- Patient's skin should be observed frequently due to individual sensitivity and susceptibility to cold.
- When filling the Cryo/Cuff, do not raise the cooler higher than 5 inches (13 cm) above the cuff to avoid excessive pressure.
- Reduce pressure with any sense of discomfort, numbness or tingling of the limb.
- The Cryo/Cuff is designed for single patient use only and may be used on the same patient for the length of treatment. The device should be cleaned if it is used on the same patient for an extended period of time (see "Cleaning Instructions" for cleaning information).
- Federal law restricts this device to sale by or on the order of a licensed healthcare professional.

APPLICATION INFORMATION:

1. Prepare cooler - Connect blue tube to cooler. Add water to line inside cooler, then add ice. For 6 to 8 hour treatment, add ice to top of cooler. Lay the insulation disk on top of ice. Attach cooler lid snugly. Allow 5 minutes with occasional shaking to chill water.

2. Apply with EMT/Cuff - At first, it is helpful to have another person assist with application of the cuff. Attach the main straps around torso securing in front and back (Fig. 1). Secure underarm strap (Fig. 2). Secure front and back elastic straps (Fig. 3). The elastic straps can be opened during shoulder exercises.

3. Fill cuff - Connect blue tube to cuff. Open air vent on cooler lid and raise cooler so that bottom of the cooler is even with top of the cuff (Fig. 4). (If the cooler is raised more than 5 inches (13 cm) above the top of the cuff, the cuff will increase in volume and weight.) Hold raised cooler for about 1 minute while cuff fills. Close air vent on cooler lid. If not using an Aircast AutoChill System, the cooler can now be disconnected from cuff by pressing the metal tab on the quick-disconnect while cooler is raised.

4. Recill water - At first, recill the water in the cuff after 15 or 30 minutes, then once an hour or as needed. Reconnect the blue tube to cuff. Lower the cooler below cuff, and the warmed water will drain from the cuff into the cooler. Allow a minute or two for the water to mix with the ice and recill, then raise the cooler and repeat the filling process (see step 3).

CLEANING INSTRUCTIONS:
After use, completely drain water from cuff, tube, and cooler. (To drain tube, elevate tube while pressing tip of quick-disconnect.) Store cooler with top off to allow drying. Periodically clean cuff, tube, and cooler using a few ounces of liquid soap added to hot water in cooler. Recycle soap/water mixture between cooler and cuff a few times, then repeat with warm water only. Rinse completely.

WARNING: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

For single patient use only.

RX ONLY

LATEX FREE

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

CAUTION: FEDERAL LAW (U.S.A.) RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PROFESSIONAL.

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